

Fitting Instructions

The wheelchair Skis are supplied pre-configured for the 15cm Ergo Lightweight front wheels. The metal brackets are located to provide good clearance between the rear of the ski and the rear wheel. The locations can be changed by squeezing the metal bracket.

1. Large metal bracket located in hole position 1 from the front, small metal bracket located in hole position 4 from the rear.

2. The elasticated cord is attached from the front of the ski to the small bracket at the rear.

3. To fit a ski

- lift the large metal bracket into an upright position at the front
- hold the front of the ski firmly on the ground (e.g. put your foot on it)
- pull back the small bracket - be careful as the elasticated cord is under tension
- drop the wheelchair front wheel between the elasticated cord and release the bracket to locate at the top of the wheel
- raise the large metal bracket against the front of the wheel and fit the cinch strap around the bracket and frame

4. To remove a ski

- undo and remove the cinch strap
- open the large metal front bracket
- hold the front of the ski firmly on the ground (e.g. put your foot on it)
- pull back the small rear bracket away from the wheel
- lift the wheel out

